


CLARUS 1.1.2

Software Update


Workflow Enhancements

- ✓ Dark mode
- ✓ Re-organised dashboard and settings menu
- ✓ Saving Red, Green & Blue colour channels
- ✓ Quick compare
- ✓ Precision Focus – available for CLARUS 500
- ✓ Optimized sorting of Fluorescein angiography- available for CLARUS 700

Dark Mode

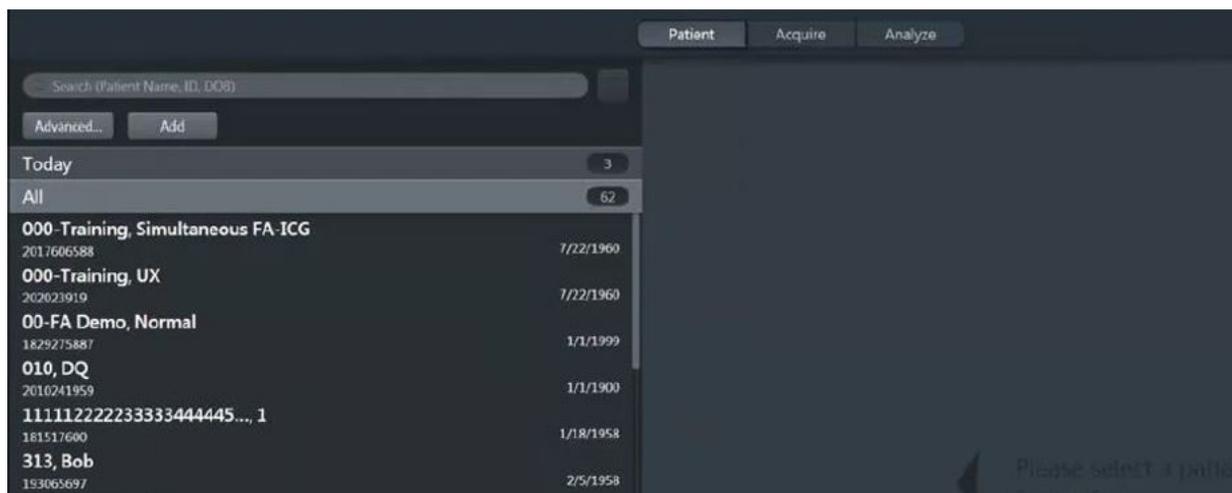

- ✓ Choose between light and dark themes. Select dark/light theme from settings, under general settings
- ✓ Better visualisation in low light environments

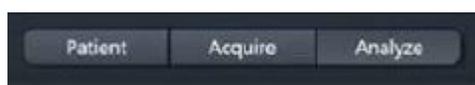

- ✓ On CLARUS 500- Patient directory, acquisition and analyse now available from the top toolbar on each section


Re-organised dashboard and settings menu




✓ Re-organised dashboard with active status update, and push notifications

✓ Re-organised settings menu with easier access and new tabs across each page


✓ New icon for dashboard

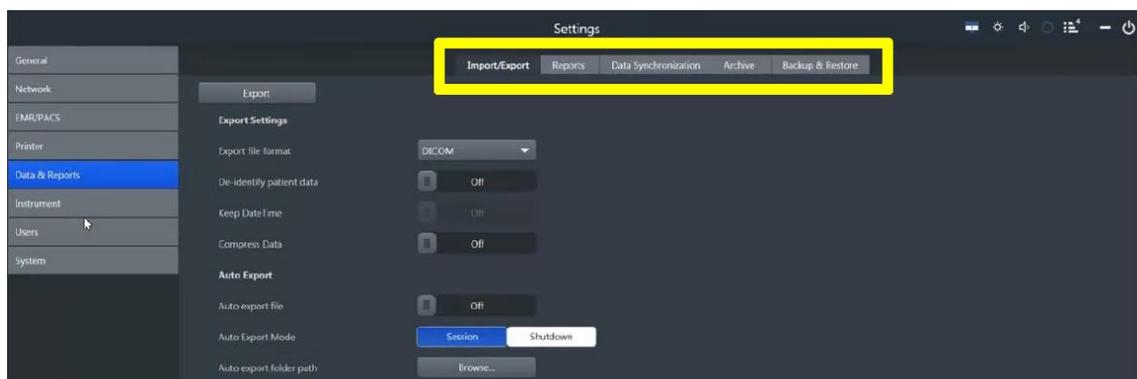


Saving Red, Green & Blue channels

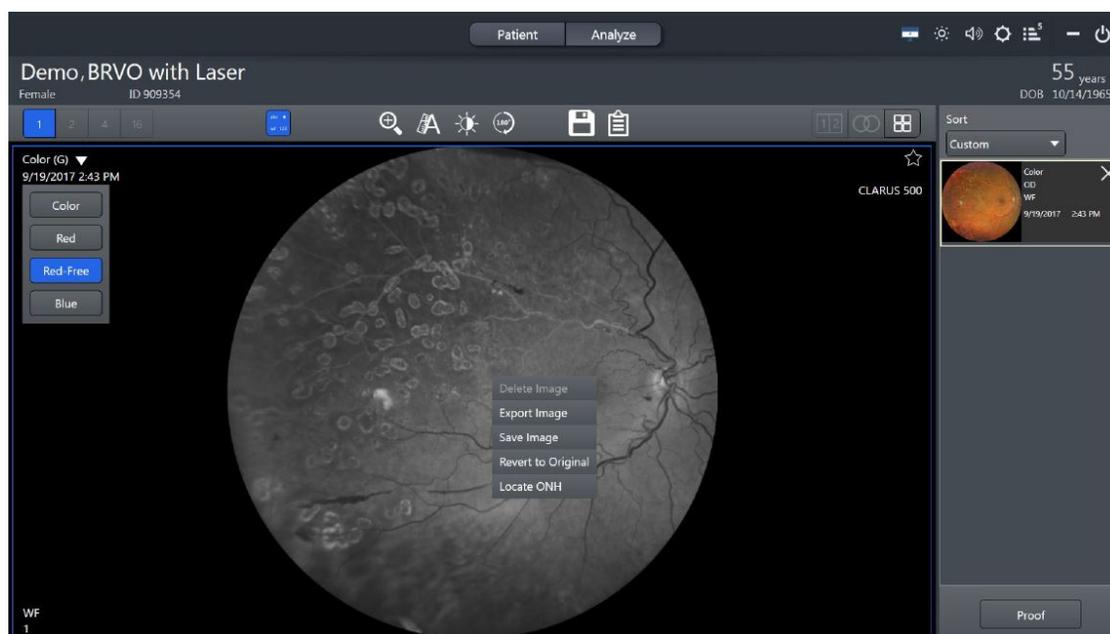

- ✓ Colour channels can be saved separately on both CLARUS and FORUM
- ✓ Select the channel and right click to save
- ✓ Image will be saved as a separate image

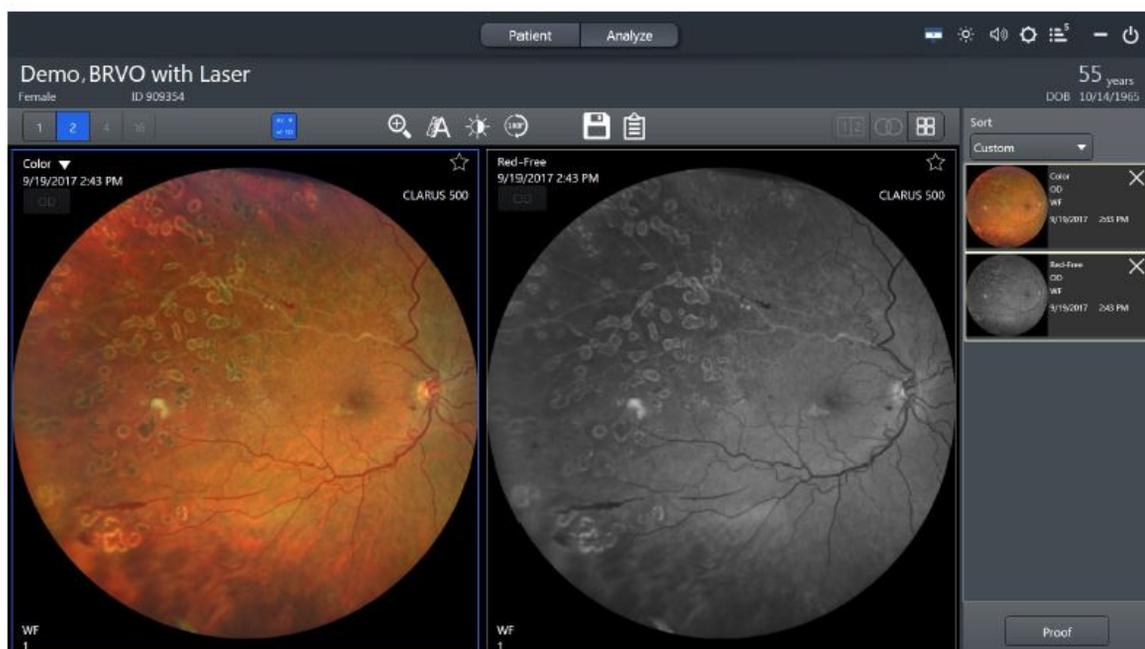

Saving

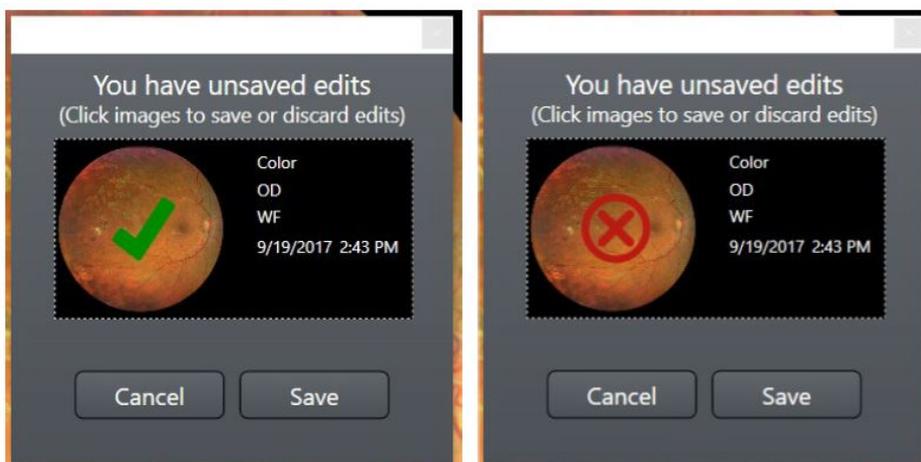

- ✓ Changed appearance of saving toggle



Precision Focus- New for CLARUS 500




- ✓ Touch anywhere on the screen to focus.
- ✓ Enables users to fine focus in a particular area of interest

AutoFocus

- ✓ Balanced multi-point AutoFocus has a constant focus across the whole field of view
- ✓ Balanced multi-point AutoFocus is optimized for Angiographies on the CLARUS 700
- ✓ 7 Fixation points

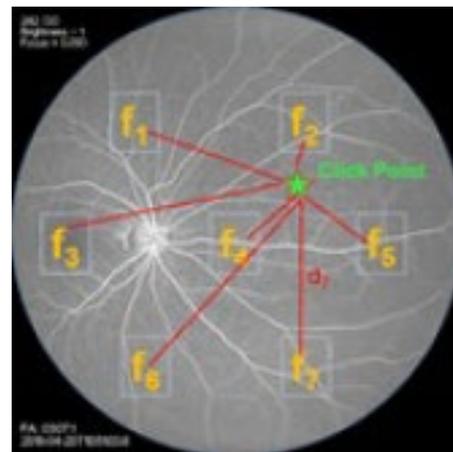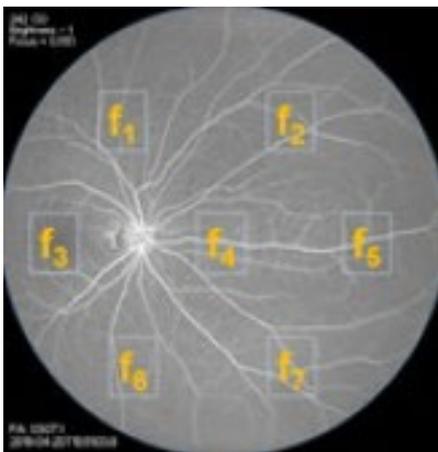

New icons

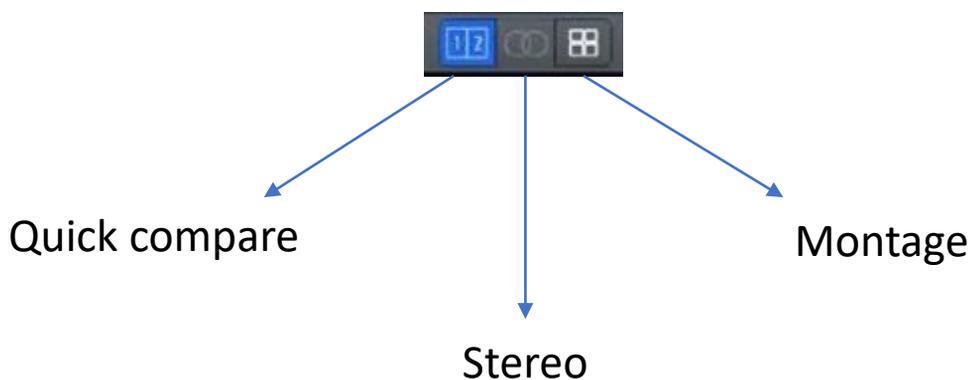

Quick compare



- ✓ Useful for follow up visits
- ✓ Easily place images side by side
- ✓ Bin 1 contains images from latest visit
- ✓ Bin 2 contains images from previous visits




- ✓ In proof sheet right click the image and select 'quick compare'

- ✓ In review select





Fluorescein Angiography- For CLARUS 700

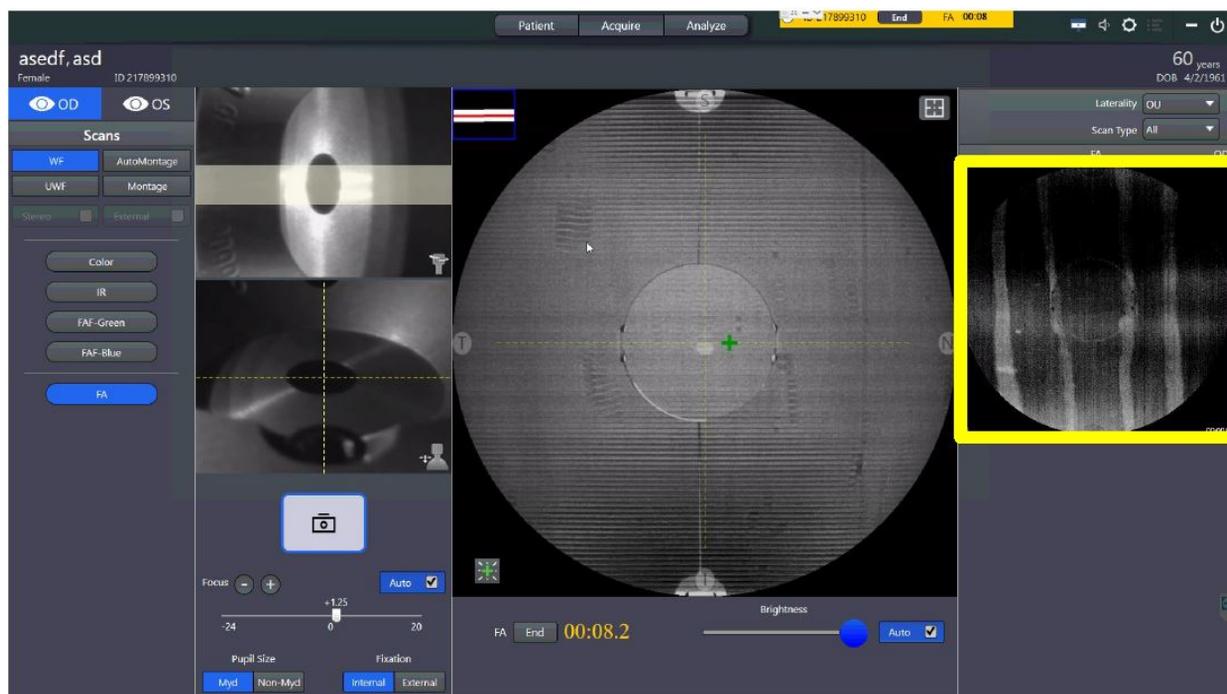

- ✓ The FA stack will remain at the top on the right hand side of the screen even if multiple different type of images are acquired

Fluorescein Angiography- For CLARUS 700

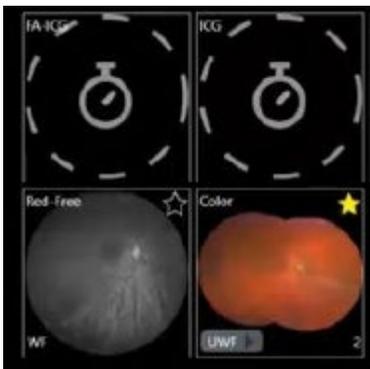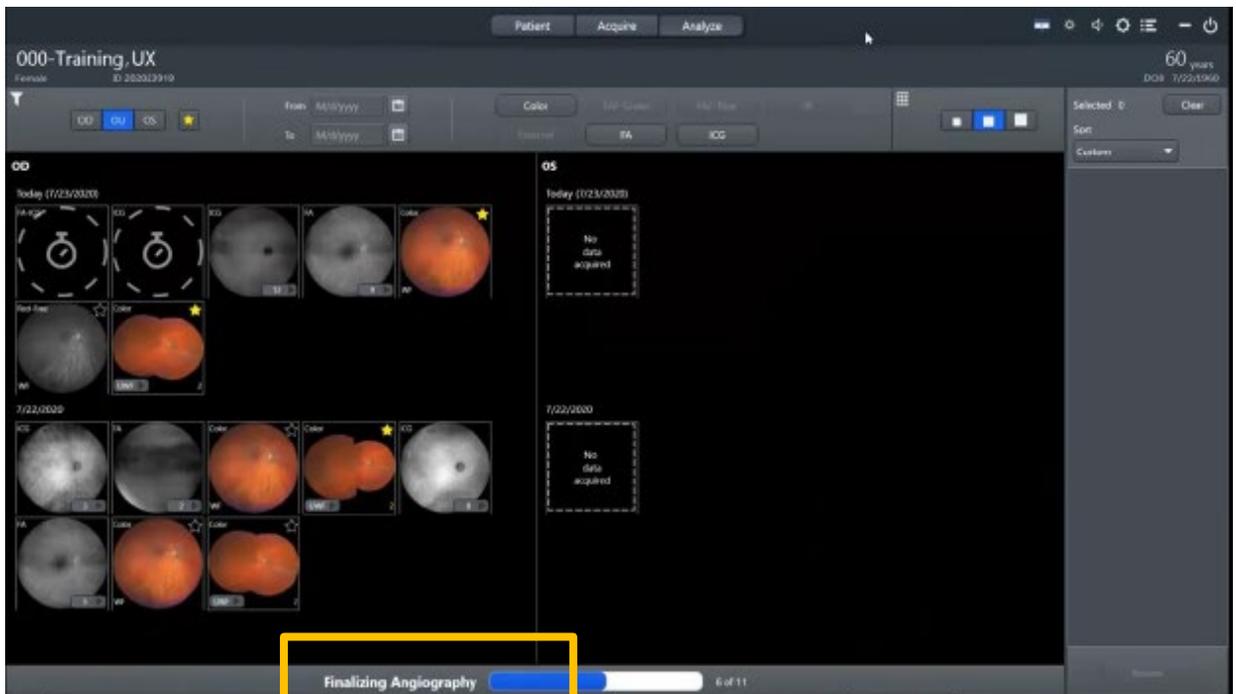

- ✓ The FFA images are only visible after the AutoBright process is complete
- ✓ Ending the FFA timer starts the AutoBright processing
- ✓ A loading bar will appear on the bottom of the analysis screen as the process completes

Fluorescein Angiography- For CLARUS 700



Sorting

- ✓ Angiography images can now be viewed in order of capture
- ✓ FFA images can still be sorted by time, laterality and modality

Expanding the stack

- ✓ No need to right click to expand angiography sequence
- ✓ Click instead on the button on the bottom right corner of the sequence thumbnail

